PL	Zarękawki foliowe Deklaracja zgodności UE Instrukcja używania środka ochrony indywidualnej	2
EN	PE sleeve covers EU Declaration of conformity Instruction for use of personal protective equipment	4
DE	Ärmelschützer EU-Konformitätserklärung Gebrauchsanweisung für die Persönliche Schutzausrüstung	6
FR	Manchons PE Déclaration de conformité UE Instructions pour l'utilisation des équipements de protection individuelle	8
CS	Ochrana na rukávy EU Prohlášení o shodě Návod na použití osobního ochranného prostředku	10
IT	Coprimaniche in pellicola Dichiarazione di conformità EU Istruzione per l'uso di dispositivi di protezione individuale	12
HU	Alkarvédő EU Megfelelőségi nyilatkozat Az egyéni védőeszközök használatára vonatkozó utasítás	14
RO	Mâneacă de protecție Declarația de conformitate UE Instrucțiuni de utilizare a echipamentului individual de protecție	16
ES	Protectores de mangas PE Declaración de conformidad EU Instrucciones de uso del equipo de protección personal	18

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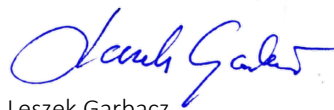
Declares under its sole responsibility that product:

Product name	Product description	Colour	Reference numbers
PE sleeve covers	PE sleeve covers, for single use, non-sterile	blue	W804100000_0028
		white	W804300000_0028

classified as Personal Protective Equipment Category I – for minimal risk, comply with provisions of Regulation (EU) 2016/425 of the European Parliament and the Council of 9 March 2016 on Personal Protective Equipment and resolution of the Council Directive 89/686/EEC and European standards: EN ISO 13688:2013/A1:2021.

Date and place of issue:
20.12.2023, Kraków

Signed on the behalf of the Manufacturer:

Leszek Garbacz
Regulatory and Documentation Manager
PRRC

INSTRUCTION FOR USE OF PERSONAL PROTECTIVE EQUIPMENT

PE sleeve covers

The instruction below should be used in conjunction with detailed information on the packaging.

Description of the product

PE sleeve covers (for minimal risk only), from polyethylene foil, standard welded, non-sterile, for single use.

Colours : blue, white
 Quantity in packaging : 100 pcs.
 Shelf life : 5 years from manufacturing date

Storage instructions

Do not expose to direct sunlight or sources of fire. Store in a dry and cool place, at a temperature of 5-30°C.

Intended use

These are PE sleeve covers, from polyethylene foil with standard welded, non-sterile, for single use. Sleeve covers are classified as a Personal Protective Equipment Category I, for minimal risk only. Sleeve covers are designed to protect against minimal risks, i.e. prolonged contact with water. Their design and labeling correspond to the requirements of the European Regulation 2016/425 on Personal Protective Equipment.

Sleeves should be used solely according to their intended application.

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Declaration of Conformity and this instruction for use available under below web address:
<https://mercatormedical.eu>

Precautions and indications for use

Wash and dry hands before taking sleeve covers out from the packaging. Before usage, inspect the sleeve covers for any defects or imperfections. If the sleeves get torn during it use, take it off and put on the new ones.

It is recommended to check that the sleeve covers are suitable for the intended use. Sleeve covers should not be used in contact with open fire. They are not intended for welding, electric shock protection, ionizing radiation or from the effect of hot or cold objects.

How to put on: take the sleeve covers out of the foil package, unfold it. Put first on your one arm then on the other.

The way to take off: take off sleeve covers. Dispose of the used sleeves

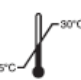
Components / hazardous components

Some sleeve covers may contain components known to be a possible cause of allergy for person allergic to them, who may develop contact irritation and/or allergic reaction. In case of an allergic reaction consult a doctor.

Disposal

Dispose of in accordance with applicable regulations.

Symbols used on the packaging

	manufacturer		date of manufacture		model		for single use only
	lot / batch number		keep dry		recyclable packaging		non-sterile
	catalogue number		keep away from sunlight		package can be treated as municipal waste		consult instructions for use
	expiry date		temperature limitation		UA sign		

PE sleeve covers

REF NO: W80410000, W80430000

0028

PPE classification & compliance

Sleeves covers are category I, for minimal risk only Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards:

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Productor: **MERCATOR MEDICAL S.A.**